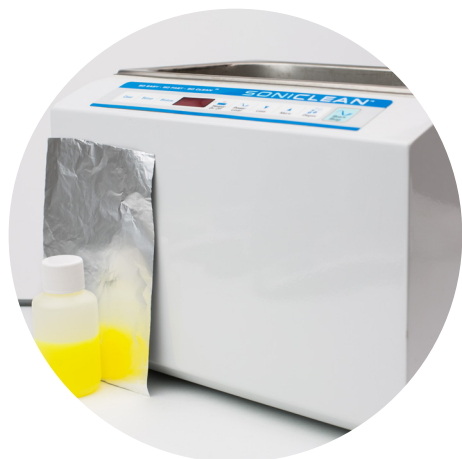

Foil Test Procedure

AS2773



Step 1

Connect to power and perform safety checks.

On a level surface, add three quarters full of fresh water with a recommended dose of detergent.

(Dose % as indicated in relevant product documentation.)

Press **degas**.

Step 2

Cut thin aluminium foil (0.025mm) to suitable size for the tank.

Press **more** or **less** to set timer for a minimum of ten seconds.

Press **start**.

Insert foil into water and solution mix, avoiding contact with tank.

Remove after ten seconds.



Step 3

Examine foil for even distribution of indents or perforations.

Document results, including temperature, time, liquid solution type and dose %.

